

Your logo and message here
We will adapt color packaging
with your logo

DRINK SPIKING PROTECTION WRISTBAND

Tests for GHB



Protect yourself



Test your drink

**1 WRISTBAND
(2 TESTS)**

HOW IT WORKS



Put 2-3 drops
on test field



Wait 10 seconds



Strong blue =
drink is spiked!



-
- + Test your drink for drink spiking
 - + Test's reliability scientifically confirmed
 - + Preventive protection
 - + Reminds to be mindful
 - + Suitable for going out, always with you
 - + Your statement for fun & safety
-

FOR YOUR OWN SAFETY

Water always shows a blue result

Tests for most popular date rape drug GHB only. GHB (Liquid Ecstasy) is odorless, tasteless and invisible in drink and can cause dizziness, loss of control, sleepiness and a blackout.

The test fields should determine gamma-hydroxybutyric acid (GHB) in realistic concentrations. A list of the beverages for which the technology is currently recommended can be found at info@twinvay.com. In addition to GHB, other substances are also suitable as Date rape drugs. There is a likelihood that there may be other substances in the drink that cannot be tested. A test should never serve as the sole basis for determining the existence of Date rape drugs. The consumption of any (un)tested drink is subject to free will. Do not consume the drink after a positive result (blue discoloration). The blue of a positive test can have different shades, intensities and mixtures. Water always colors blue. That's a security function to show, that a test field had (accidentally) contact with water and is wet, so it cannot be used for another test. The result of the wristband is only for personal use and for information purposes. Always have a beverage that tests positive professionally tested in the laboratory. Xantus makes no representations or warranties as to the legality or other consequences of disclosure of the test result or reliance on the results. All liability and warranties are disclaimed. Each field can only react once. Always pay attention to your drink and reduce alcohol consumption. Be careful and alarm your friends or staff if you feel uncomfortable. Store wristbands dry and in shade (protected from sunlight). Best is to store wristbands in polybag and keep inside the packaging.